




OPHTALMO

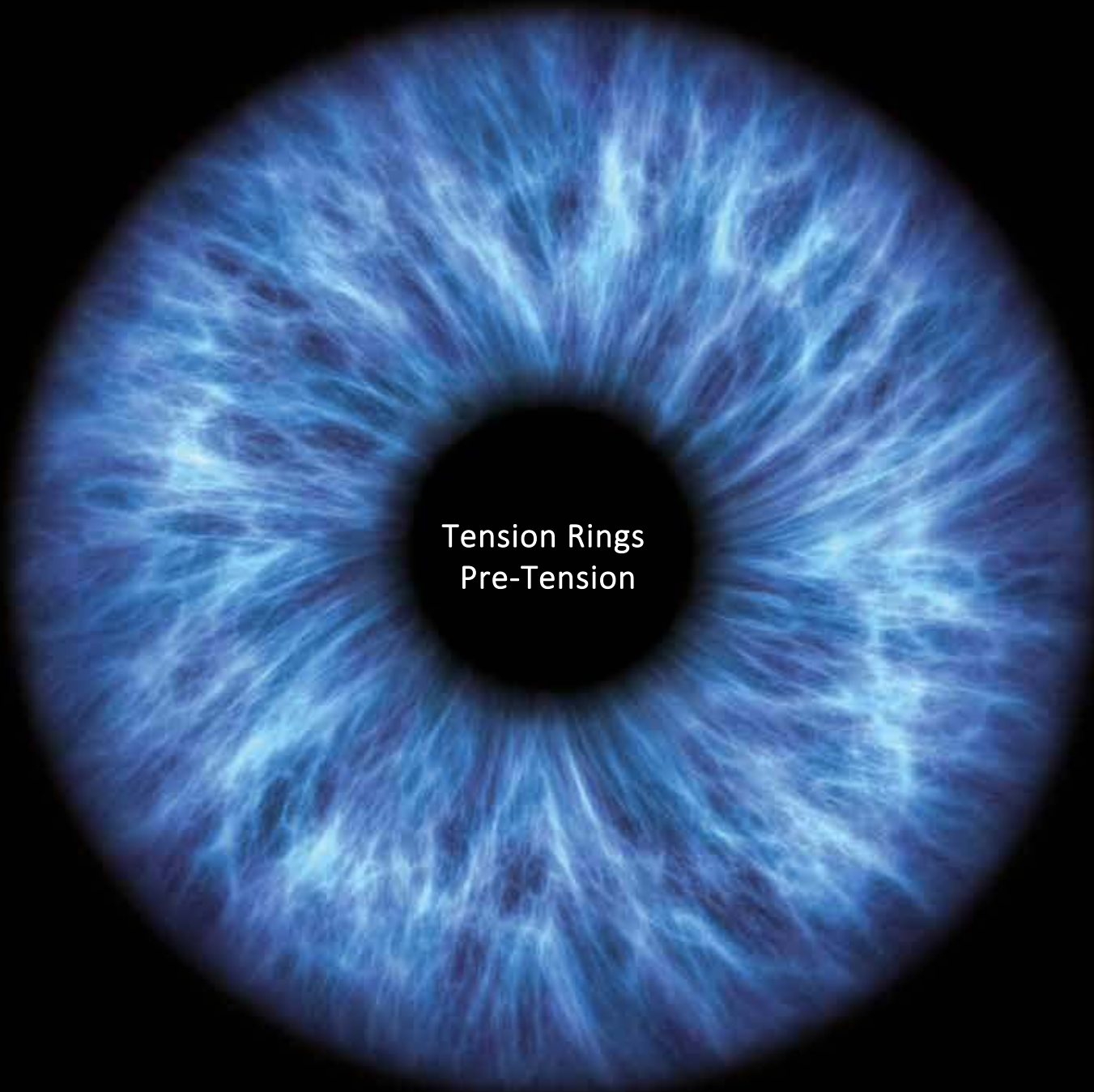

Tension Rings
Pre-Tension

THIS IS WHAT THE FUTURE LOOKS LIKE

Pre-Tension®

Preloaded Capsular Tension Ring
in a single use injector



NEW

Selfloading

Onehand-Technique

Implantation – clockwise or counter clockwise

Patented Hockey design of CTR

Implantation with 2.0 mm incision



OPHTALMO

THIS IS WHAT THE FUTURE LOOKS LIKE


Pre-Tension®

Preloaded Capsular Tension Ring in a single use injector


Push the pushrod forward until it is limited.



Slowly put the Capsular Tension Ring into the tip of Injector.

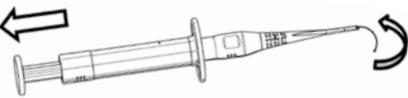

The Capsular Tension Ring is completely put into the tip of Injector.

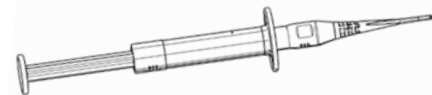


Technical Data


Model	Pre-Tension® 13/11
Compression	From 13 mm to 11 mm
Material	Clear PMMA, flexible ring
Preloaded	Self-loading, single packaging a valuable addition to your surgical armamentarium
Injector	<ul style="list-style-type: none">• Easy to use• Total control (includes action/reaction mechanism)• Implantation clockwise or counter clockwise (indicators on injector)• Long small tip (especially convenient in cases of deep set eyes)• Beveled tip (for easy entry into the incision)
Capsular Tension Ring	<ul style="list-style-type: none">• Circular expansion and stabilization of the capsular bag• Safe IOL centration in eyes with zonular dehiscence• Prevents IOL decentration after capsular shrinking• Stabilized conditions during Phaco-emulsification surgery• Reduced risk of capsular fibrosis• Improves visual acuity when implanted along with premium IOL

Contact:

Ophtalmo Service
Industriepark De Bruwaan 5
9700 Oudenaarde
Tel.: +32 (0)55.31.30.38
Email: info@ophtalmoservice.be
www.ophtalmoservice.be

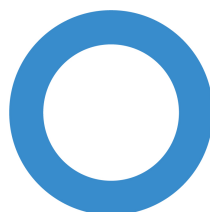

OPHTALMO
THIS IS WHAT THE FUTURE LOOKS LIKE

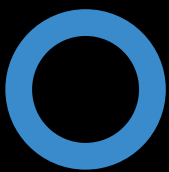

OPHTALMO
THIS IS WHAT THE FUTURE LOOKS LIKE

- SURGICAL
- CONSUMABLES
- MEDICAL DEVICE